



UK  
CA | CE

EC REP

CS Lifesciences Europe Ltd  
The Black Church  
St. Mary's Place  
Dublin 7  
D07 P4AX  
Ireland

eurep@cslifesciences.com

Rev: 2024.01

Making **cutting-edge**  
medical equipment  
**affordable**

**Head Office**

Alerta Medical  
4 Symington Place  
Riverside Business Park  
Irvine  
KA11 5DE  
United Kingdom

[www.alertamedical.com](http://www.alertamedical.com)

# Instruction Manual



**W-FBCT**  
Wireless Transmitter




UK Version

The **Alerta Wireless Transmitter** is for use with Alerta Wireless Fall Prevention products and pairs wirelessly with the Alerta Wall Point Receiver.

Alerta Wireless Transmitter is for use with Alerta's Wireless; Chair Alertamat, Bed Alertamat, Deluxe Carpet Alertamat, Floor+ Alertamat, WetSense Bed & Chair Alertamats and Alertamat Crashmat.

It uses radio frequency technology to connect to Alerta Wall Point Receiver with a wireless range of up to 95 meters. Once paired with Alerta Wall Point Receiver it will only transmit to that Wall Point Receiver and will not interfere with other Alerta Wall Point Receivers in use nearby.

## Diagrams

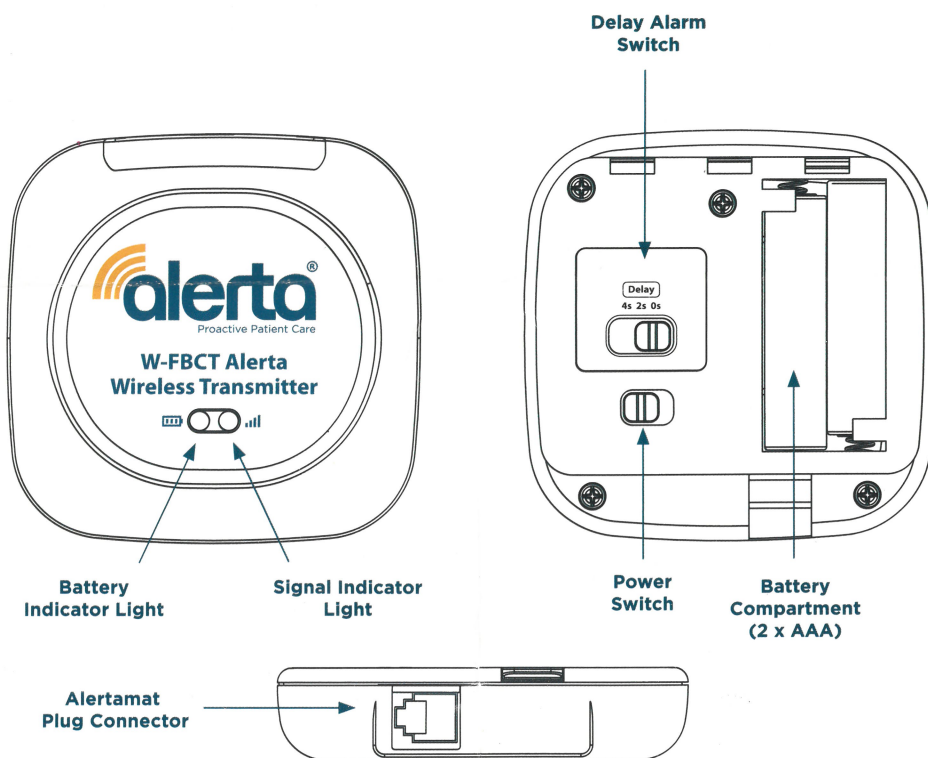

## Set-up Instructions

1. Remove rear case using the release clip and install 2 x AAA batteries with the positive (+) and negative (-) ends facing in the proper direction.
2. Turn the power on. Battery is normal if the indicator light on the front blinks twice after turning the device on. If the indicator light keeps blinking in red, it indicates battery level is low, and the batteries need to be replaced.
3. The transmitter has the functionality to delay the alarm sounding for Bed and Chair Alertamats, see 'Delay Alarm Switch'. This means after activation the alarm will not sound for 0, 2 or 4 seconds on Bed and Chair Alertamats, (does not apply for other devices). If you wish to have a delayed alarm, now is the best time to switch it to the setting you require.
4. Now replace the rear cover.
5. Have Alertamat (Alerta Wireless Chair Alertamat, Bed Alertamat, Deluxe Carpet Alertamat, Floor+ Alertamat, WetSense Bed / Chair Alertamat or Alertamat Crashmat) to hand and insert the plug into the Alertamat Plug Connector in the transmitter. Place Alerta Wireless Transmitter in the pocket of the Alertamat and seal the pocket closed. Test the Alertamat before use.
6. Now please refer to W-WPR Alerta Wall Point Receiver User Manual to pair this transmitter wirelessly with the Wall Point Receiver.

## Operation

**Low battery status** - The battery indicator will blink in red every 3 seconds with a "beep" sound. Please replace the batteries to avoid malfunction.

**Delay Alarm Switch** - If you wish to have a delayed alarm, switch 'Delay Alarm Switch' to the setting you require, 0, 2 or 4 seconds. Please note this delay function only applies to Bed and Chair Alertamats.

**Storage** - Remember to switch Alerta Wireless Transmitter to 'OFF' if not in use for a period of time.

Status	Indicator
ON	Red, green light blink twice
Stand by	Red, green light stay off
Sending wireless signal	Blinks in green when battery is normal. Blinks in red when low battery
Low battery	Blinks in red slowly
Battery life	Only for reference (Usage & distance dependant)
30 transmission/day	12 months
100 transmission/day	9 months

## CAUTION

Improper installation of the battery or the installation of the incorrect battery type can result in damage to the contacts and cause improper operation.